

WARMING PAD

Warming Pad

Benefits:

Our warming pad is CSA/UL listed, giving you confidence in the safety, security and functionality of our thermal support. It's designed to reduce risk of hypothermia in post-operative patients, who are shown to be more susceptible following anesthesia.

Features:

- Built in safety. The enclosed heating element warms up in about 15 minutes, maintaining the factory-set temperature during operation.
- The controller offers three timed cycle setting options-60, 90 and 120 minutes.
- The alarm helps monitor patients, sounding an alarm at five minutes until the end of the selected time cycle. Low-tone beeps warn the operator before the pad shuts off completely.
- If the Warming Pad becomes unplugged or exceeds the safety temperature, alert mode will sound. The pad will become a cool down process and a red light will illuminate to warn the operator.
- Controller mounts on the cage door.
- Specs: 20" W x 25" L, power cord is 10'. Electrical: 120/240 VAC, 50/60 HZ, .5 A at 120 VAC, .25 A at 240 VAC.



912.1000.05 Warming Pad, 120V 10 lbs.
912.1000.06 Warming Pad, 240V 10 lbs.