



A supplement for:
The Scale Display

SHOR-LINE
SCHROER MFG. CO.

⚠WARNING This product is for animal use only. We do not condone this product for other uses.

{Save This Supplement For Future Reference}


PN: 006.3312.01
Updated: 08.01.14

Operation of the K9W8 Scale Display

THIS PRODUCT IS CALIBRATED AT THE FACTORY.

Calibration is not required at initial set-up.

For questions, contact Shor-Line Technical Service at 800.444.1579 option 6.

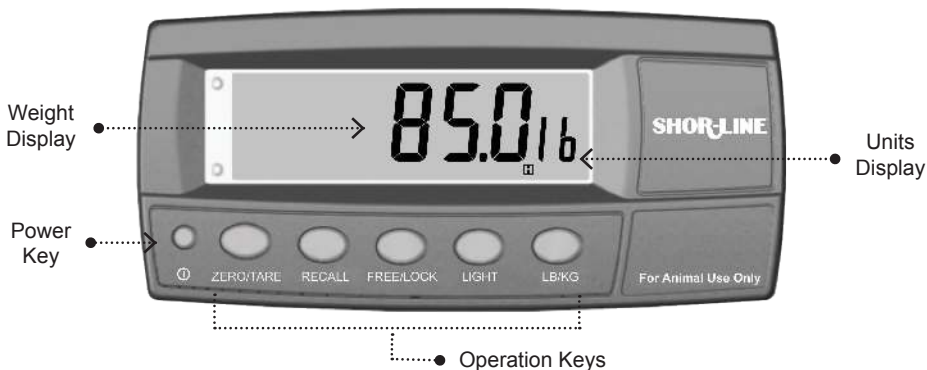

{Power}

The <POWER> key is used to turn the instrument on and off.

- **To initially turn the instrument ON:** Press and hold the <POWER> key until the display starts up.
- **To Turn the instrument OFF:** Press and hold the <POWER> key for three seconds. The instrument will display **OFF** followed by the three-second countdown.

{Zero/Tare}

The <ZERO/TARE> key is used to perform a Zero adjustment with 100% range on the scale display when an empty scale has drifted away from a true zero reading.

- The Zero adjustment is stored when power is removed and is re-used when next powered up.
- **Long Press** of the <ZERO/TARE> key will remove any stored zero adjustment.

{Recall}

The <RECALL> key will recall the previously HELD weight (manual or automatically generated) for 5 seconds. The <RECALL> function will time out 2 minutes after the last animal weighment.

{Free/Lock}

The <FREE/LOCK> function controls the weighing algorithm that analyses the weight data of a moving animal and calculates a HELD or LOCKed weight.





- Long Press: A long press of the <FREE/LOCK> key will toggle the automatic HOLD function On/Off.
- LOCK = Automatic HOLD: As soon as an animal is placed on the scale the instrument will flash the "H" annunciator and automatically capture a stable weight. When captured the "H" flashing stops and the weight is held (LOCKed) on the screen until the animal is removed. A short press of the <FREE/LOCK> key forces a recalculation of the animal weight. After the animal is removed from the scale the weight will return to within the zero band and a normal weight reading of 0.00 will be displayed.
- FREE = Manual HOLD: In the FREE mode a short press of the <FREE/LOCK> key will capture the instantaneous displayed weight as the HELD weight. This is useful for operators that prefer to use their own judgment as to when to capture the weight manually. As with the automatic hold function when the weight returns to within the zero band a normal weight reading of 0.00 will be displayed.

{Light}

- In AC Mode: Used to adjust backlight intensity (Off/Lo/Hi). "Hi" intensity will be the default.

{LB/KG}

Press and hold the <LB/KG> key to toggle units of measure from Lb. to Kg.

Display Symbols		
	Zero	Visible when the gross reading is within $\pm \frac{1}{4}$ of a division of true zero.
	Motion	Visible when the displayed reading is not stable.
	Zero Band	Visible when the displayed weight is within the zero 'dead' band setting. (The zero band symbol shows near the top right corner of the display.)
	Hold	Visible when the displayed reading is held.

SHOR-LINE[®]

SCHROER MFG. CO.

{Shor-Line}

Schroer Manufacturing Company
511 Osage Ave.

Kansas City, Kansas 66105, USA

phone 800.444.1579

fax 913.281.5339

email guides@shor-line.com

www.shor-line.com

{Shor-Line Limited}

Vale Business Park

Llandow,

Vale of Glamorgan CF71 7PF

United Kingdom

phone +44 1446 77 20 41

fax +44 1446 77 36 68

email quality@shor-line.co.uk

www.shor-line.co.uk