

SCALE CALIBRATION

OPTION 1

1. Press and hold POWER and LB/KG until FULL SETUP displays. Proceed when the screen says FILTER.
2. Press and release (do not hold) ZERO/TARE until ZERO displays.
3. Press RECALL 1 time.
4. Press FREE/LOCK 1 time and wait until 0.00 displays and flashes. (This will take a few seconds. Don't continue until 0.00 flashes.)
5. Press ZERO/TARE 2 times. The screen will display SPAN.
6. Press RECALL 1 time.
7. Press FREE/LOCK 1 time.
8. Press FREE/LOCK to move digit to be changed (flashing).
9. Press LIGHT to change digit (Set to 20 lbs)
10. Add 20 lb Calibration weight and wait 5-10 seconds.
11. Press LB/KG 1 time.
12. Press and hold POWER and LB/KG until SAVING displays.

