

**BLUE-LINE**  
pure innovation

*A guide for:*

## Blue-Line Fold-Up Wall Mount Table

**SHOR-LINE®**

**⚠WARNING** This product is for animal use only. We do not condone this product for other uses.

**⚠WARNING** To reduce the risk of injury, the user must read and understand this guide before installation of this product.

{Save This Guide For Future Reference}

PN: 006.1530.01  
Updated: 10.03.2014



# Introduction

Thank you for purchasing Shor-Line products.

We hope this guide answers any questions you might have in regards to use, care or installation. If you can't find the answers to your questions, have further questions or would like to purchase additional products, please contact us at 800.444.1579 or 913.281.1500. If in Europe phone +44 1446 77 20 41.

Consider keeping this guide in a safe and convenient place for future assistance.


**IMPORTANT:** DO NOT ATTEMPT TO DISASSEMBLE TABLE WITHOUT CONTACTING SHOR-LINE.

*For your safety, please follow the instructions on these labels:*

## ▲CAUTION ▲WARNING

**▲WARNING** This product is for animal use only. We do not condone this product for other uses.

**▲WARNING** To reduce the risk of injury, the user must read and understand this guide before installation of this product.

**WARNING** **California Proposition 65 Warning**  
*This product can expose you to chemicals including nickel (metallic), which are known to the state of California to cause cancer.  
For more information go to: [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)*

{Shor-Line}  
511 Osage Ave.  
Kansas City, Kansas 66105, USA  
phone 800.444.1579 fax 913.281.5339  
email [guides@shor-line.com](mailto:guides@shor-line.com)  
[www.shor-line.com](http://www.shor-line.com)

{Shor-Line Limited}  
Vale Business Park  
Llandow,  
Vale of Glamorgan CF71 7PF  
United Kingdom  
phone +44 1446 77 20 41 fax +44 1446 77 36 68  
email [quality@shor-line.co.uk](mailto:quality@shor-line.co.uk)  
[www.shor-line.co.uk](http://www.shor-line.co.uk)

### ▲CAUTION

Do not mount this table to walls with metal studs unless sufficient wall reinforcement has been installed by a qualified mechanical contractor. For assistance, contact Shor-Line with details of your installation.

### ▲WARNING

Shor-Line products are designed for animal use only. Never allow a person on your Blue-Line Fold-Up Wall Mount Table.

### ▲CAUTION

The Blue-Line Fold-Up Wall Mount Table is spring loaded to counterbalance the table from the horizontal position to the full up position. Mount table to wall fully assembled. Do not attempt to disassemble table without consulting Shor-Line.



## {Contents}

### **Section 1: Unpacking & Inspecting Shipment**

Unpacking & Inspecting Steps .....	4
Features of the Blue-Line Fold-Up Wall Mount Table, Lateral.....	5
Features of the Blue-Line Fold-Up Wall Mount Table, Peninsula.....	6

### **Section 2: Replacement Parts**

Replacement Parts .....	7
-------------------------	---

### **Section 3: Installing the Blue-Line Fold-Up Wall Mount Table**

Before Starting .....	8
Mounting Instructions .....	8

### **Section 3: Leveling Your Blue-Line Fold-Up Wall Mount Table**

Adjustment Instructions.....	11
------------------------------	----

### **Section 4: Blue-Line Polyurethane Edge Clean & Care**

Appearance & Life of the Polyurethane Edge .....	12
Polyurethane Edge Care & Cleaning Best Practices .....	12

### **Section 5: Stainless Steel Care & Cleaning**

Quality & Life of Your Product .....	13
Care & Cleaning Best Practices.....	13
Protecting Against Disease & Infection .....	14

### **Section 6: Services, Terms & Conditions, Warranty**

Terms & Conditions .....	15
--------------------------	----

• Blue-Line Fold-Up Wall Mount Table, Lateral - Standard Blue	Item # 903.1530.04
• Blue-Line Fold-Up Wall Mount Table, Lateral - Jade	Item # 903.1530.05
• Blue-Line Fold-Up Wall Mount Table, Lateral - Taupe Gray	Item # 903.1530.06
• Blue-Line Fold-Up Wall Mount Table, Lateral - Black	Item # 903.1530.07
• Blue-Line Fold-Up Wall Mount Table, Peninsula - Standard Blue	Item # 903.1531.04
• Blue-Line Fold-Up Wall Mount Table, Peninsula - Jade	Item # 903.1531.05
• Blue-Line Fold-Up Wall Mount Table, Peninsula - Taupe Gray	Item # 903.1531.06
• Blue-Line Fold-Up Wall Mount Table, Peninsula - Black	Item # 903.1531.07



## Section 1: Unpacking & Inspecting Shipment

---

### {Unpacking & Inspecting Steps}

#### **Step 1**

Locate the Packing Slip that describes the parts and quantities shipped. Your Blue-Line Fold-Up Wall Mount Table should be carefully examined upon arrival for possible damage in shipment. If any damage is found, notify Shor-Line immediately.

Your Blue-Line Fold-Up Wall Mount Table comes fully assembled.

#### **Step 2**

Unpack and place in desired position in facility. This location must have reimbraced wooden studs in the walls.

#### **Step 3**

Review Section 6: Shor-Line's Warranty Information. For more information visit: **[www.shor-line.com](http://www.shor-line.com)**

#### **Step 4**


Be sure that all parts are contained for your Blue-Line Fold-Up Wall Mount Table.

#### **Step 5**

Check content for concealed damage.

Call Shor-Line immediately so we can expedite replacements or repairs.


 {Features of the Blue-Line Fold-Up Wall Mount Table, Lateral}

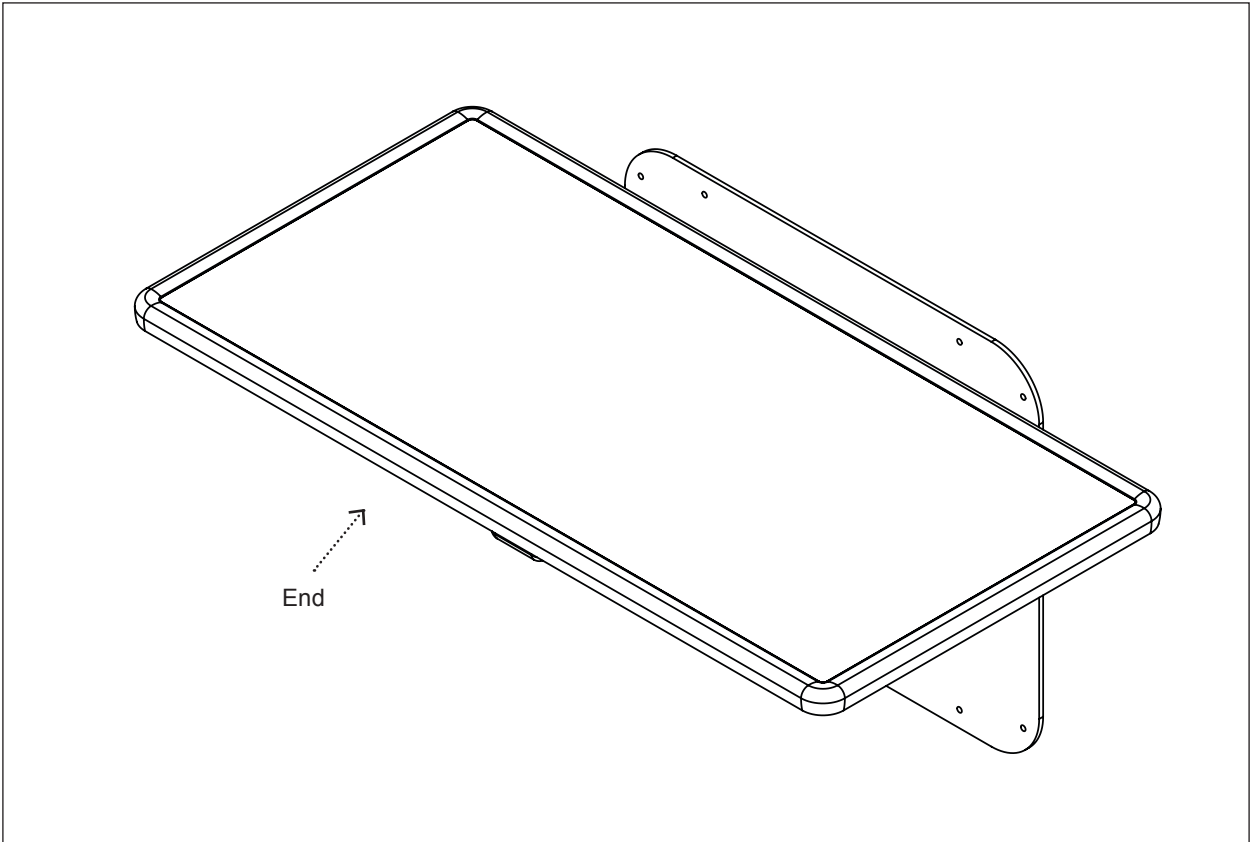

**Blue-Line Fold-Up Wall Mount Table, Lateral**  
Items: 903.1530.04 - .07

Stainless Steel Blue-Line Table Top is 44"L x 22"W x 1"H.
From wall to end of table is 24". Actual table is 22".
Blue-Line Table Top with radius corners and laminated polyurethane edging holds up to one liter of fluid.
New torsion spring technology allows you to easily and safely raise and lower the table.
Leveling adjustment with lock.
Weight capacity—200 lbs. when installed based on manufacturers recommendations.
5.5" from the wall when folded up.
Table is easily mounted to any standard 2" x 4" reinforced wooden stud wall with 16" stud centers. Also compatible with our Stainless Steel Utility Chase.
Table mounting template included (not shown). 006.1130.04.01




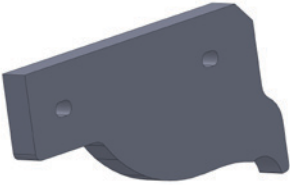
**{Features of the Blue-Line Fold-Up Wall Mount Table, Peninsula}**

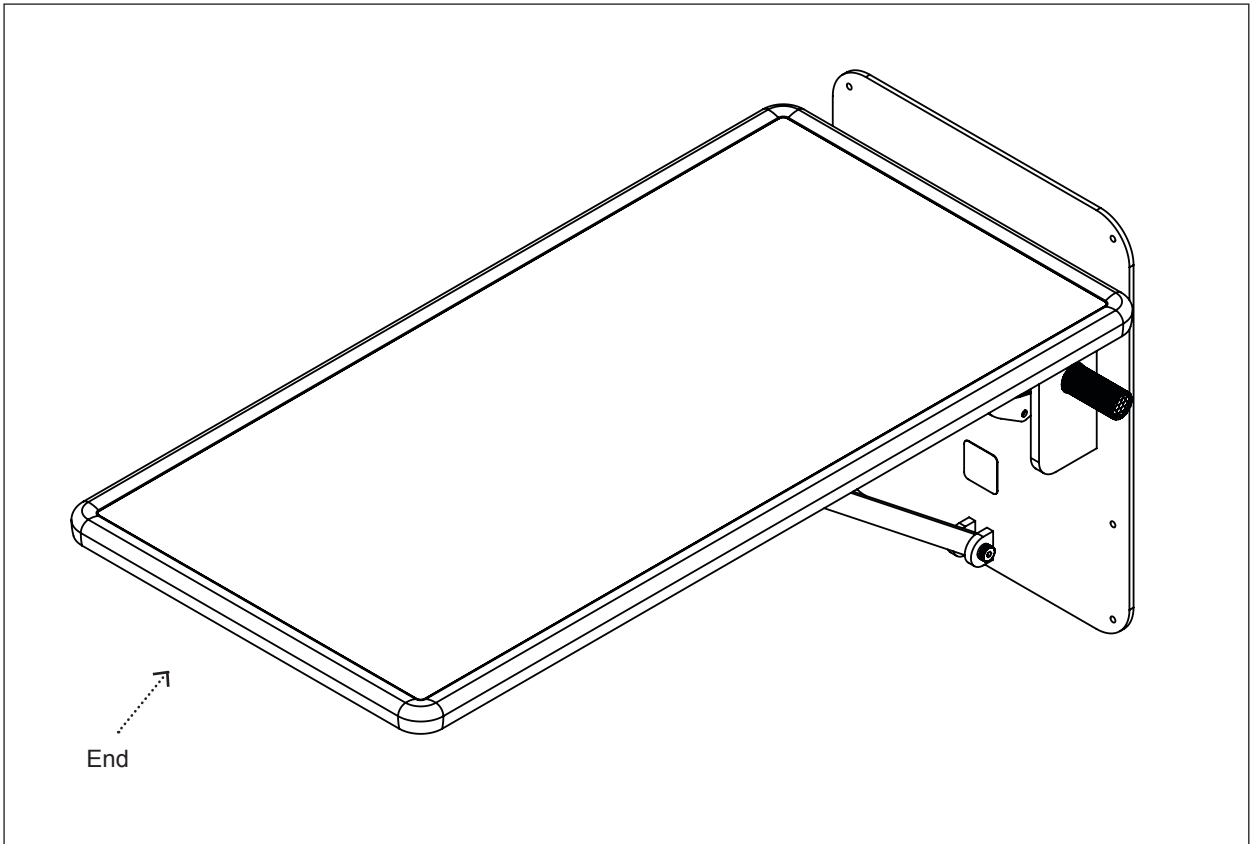

**Blue-Line Fold-Up Wall Mount Table, Peninsula**  
**Items: 903.1531.04 - .07**

Stainless Steel Blue-Line Table Top is 44"L x 22"W x 1"H.
From wall to end of table is 46". Actual table is 44".
Blue-Line Table Top with radius corners and laminated polyurethane edging holds up to one liter of fluid.
New torsion spring technology allows you to easily and safely raise and lower the table.
Leveling adjustment with lock.
Weight capacity—200 lbs. when installed based on manufacturers recommendations.
5.5" from the wall when folded up.
Table is easily mounted to any standard 2" x 4" reinforced wooden stud wall with 16" stud centers. Also compatible with our Stainless Steel Utility Chase.
Table mounting template included (not shown). 006.1130.04.01

# Section 2: Replacement Parts

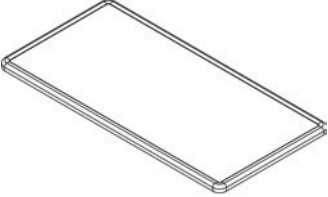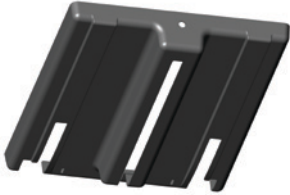Quantity Needed	Description	Part Number	
1	Trimmed Top, Blue	300.1701.01	
	Trimmed Top, Black	300.1701.02	
	Trimmed Top, Jade	300.1701.03	
	Trimmed Top, Taupe Gray	300.1701.04	
1	Bottom Cover	181.1530.00	
2	Cam Right & Left, Peninsula	193.1531.02	
2	Cam Right & Left, Lateral	193.1530.02	



## Section 3: Installing the Blue-Line Fold-Up Wall Mount Table

---

### {Before Starting}

**IMPORTANT:** This table is spring loaded to counterbalance table from horizontal position to the full up position. The table is shipped fully assembled and tested. Mount table to wall completely assembled. DO NOT ATTEMPT TO DISASSEMBLE TABLE WITHOUT CONSULTING SHOR-LINE.

### {Mounting Instructions}

#### Step 1

Select a location for table installation. This location must have a suitable reinforced frame or wooden studs.

---

**⚠WARNING** DO NOT MOUNT THIS TABLE TO WALLS WITH METAL STUDS UNLESS SUFFICIENT WALL REINFORCEMENT HAS BEEN INSTALLED BY A QUALIFIED MECHANICAL CONTRACTOR. For assistance, contact Shor-Line with details of your installation.

---

#### Step 2

Have a qualified technician locate the reinforced wooden wall studs using a stud finder. The table has mounting holes for 16" stud centers.

#### Step 3

Using the mounting template provided, carefully line up stud center lines and set the table working height at the desired level. Once height is established, use a carpenter's level to make sure the line on the template labeled "Table Working Height" is level. Attach template to the wall using tape, mark screw mounting locations on the wall and drill  $\frac{1}{8}$ " diameter pilot hole 2" deep into the wall at each screw location. (For reference, see Fig. 3A for Peninsula mode and Fig. 3B for Lateral mode.)

#### Step 4

Lift table into position.

---

**⚠CAUTION** THIS TABLE REQUIRES A TWO PERSON LIFT; DO NOT ATTEMPT A ONE PERSON LIFT.

---

#### Step 5

Mount table level and secure to wall using the 8 lag screws provided. Ensure that all screws hit the studs and are tightened completely.

#### Step 6

Test the table by raising and lowering the table. The table is factory balanced before shipment and should be close to weightless in all positions. If the table is not balanced correctly, contact Technical Services at 800-444-1579.

## Section 3: Leveling Your Fold-Up Wall Mount Exam Table

Fig. 3A




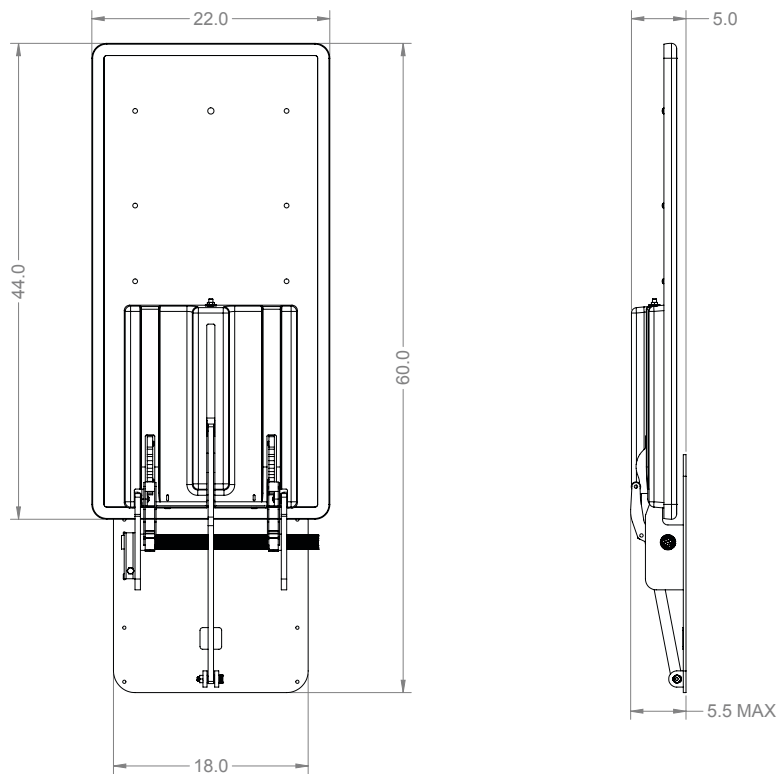


Fig. 3B

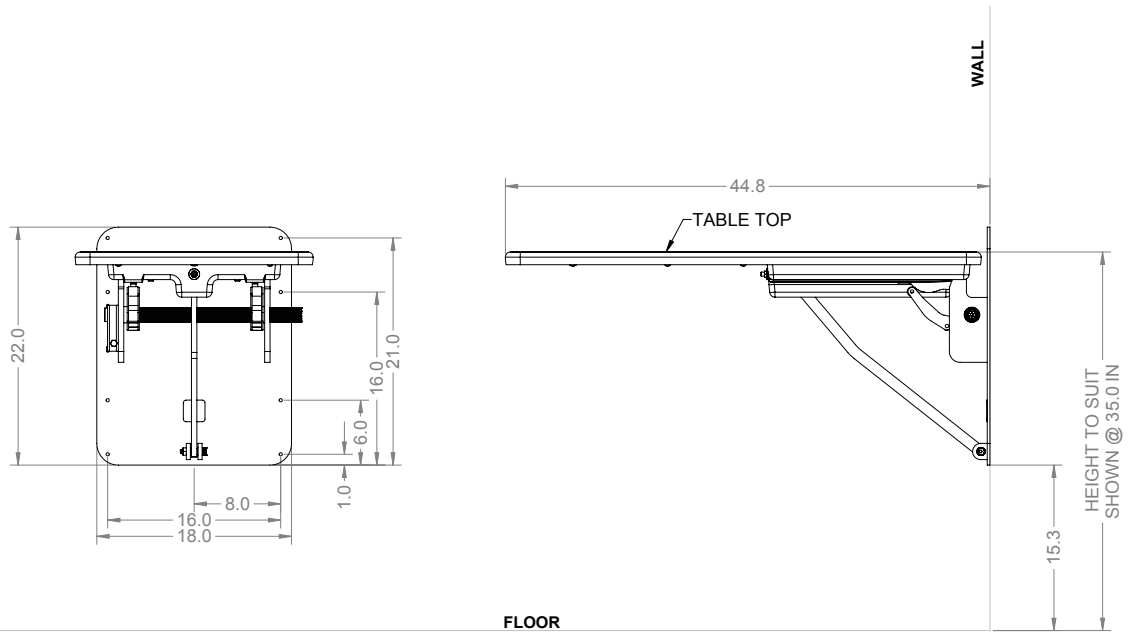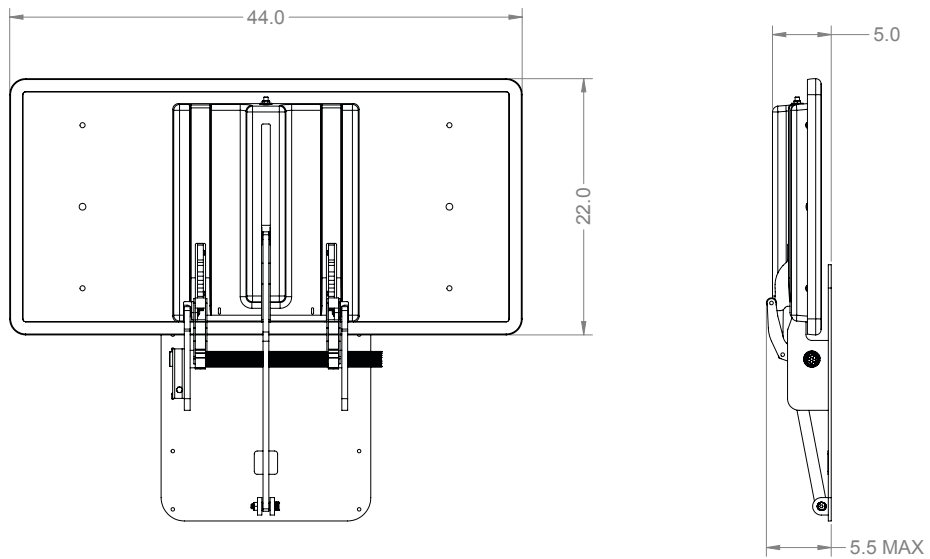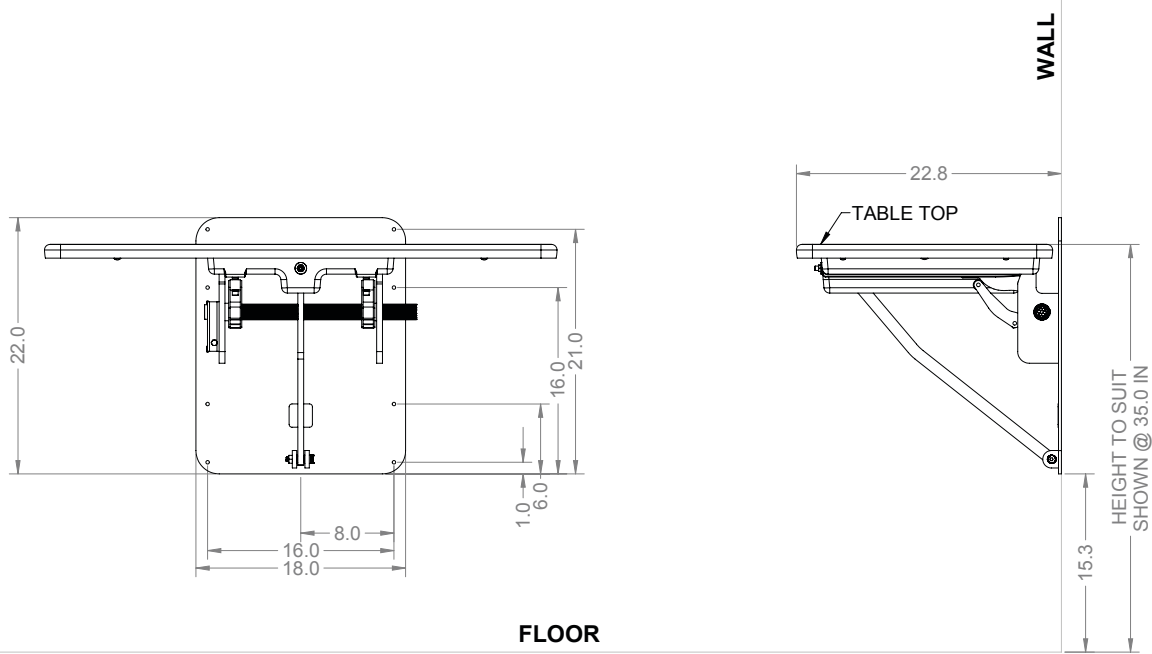

## {Adjustment Instructions}

The following steps should be taken to adjust the level of the table.

### TOOLS REQUIRED:

$\frac{9}{16}$ " Wrench

$\frac{3}{16}$ " Allen Wrench

#### Step 1

Using a  $\frac{9}{16}$ " Wrench, loosen the jam nut. See Fig. 3A.

#### Step 2

Put a level on the surface of the table, ensuring that it is perpendicular to the wall.


#### Step 3

Using a  $\frac{3}{16}$ " Allen Wrench, turn the set screw until the level bubble is centered. Turning clockwise will raise the end of the table and counterclockwise will lower the end of the table.


#### Step 4

While holding the set screw in place with the allen wrench, tighten the jam nut with the  $\frac{9}{16}$ " Wrench.

Fig. 3A

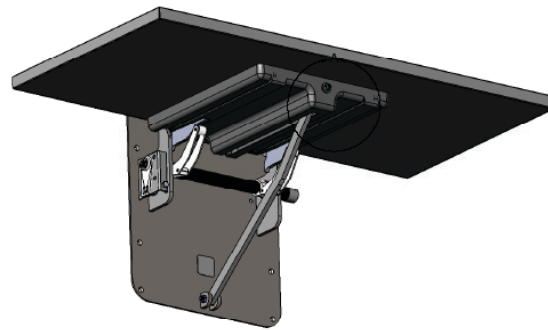

DETAIL A

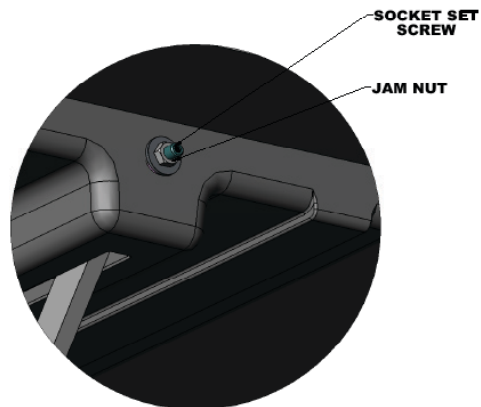



## Section 4: Blue-Line Polyurethane Edge Care & Cleaning

---

### {Appearance & Life of the Polyurethane Edge}

Polyurethane moldcast edges are generally stain resistant to most compounds or staining agents found in the office, food and health care industries.

**IMPORTANT:** Routine maintenance will ensure quality and life of your long-term investment and protect animals from transmittable disease and infection.

**IMPORTANT:** To safeguard your investment and rights to the Shor-Line's Warranty, it is important that you know factors that degrade the polyurethane edging. Permanent damage to the edge due to corrosion, pitting or discoloration can occur from:

- Bacteriostatic agents such as Iodine and Betadine have the greatest staining potential and care should be exercised to avoid exposing the edging to these.
- Harsh cleaners with abrasive additives (such as high concentrations of Bleach) have the potential to corrode or discolor the edging.
- Sharp metal objects, sandpaper or hard bristle brushes can cause pitting in the edging if used to try to remove stains.

### {Polyurethane Edge Care & Cleaning Best Practices}

**IMPORTANT:** You play an essential role in safeguarding your investment and rights to Shor-Line's Warranty. We strongly recommend the following best practices to help maintain the appearance of the polyurethane edging:

- Ordinary soiled edges caused by normal use can be cleaned with a damp cloth or sponge and Clorox® Clean-Up Cleaner with Bleach or Formula 409®.
- For stains, if a scrub brush is used, make sure it is a soft bristle type to prevent scratching.
- Successive cleanings may be required using a soft bristle brush and 70% Isopropyl Alcohol.
- Make sure all surfaces are rinsed properly.
- Follow instructions for use on all cleaning products used.

## Section 5: Stainless Steel Care & Cleaning

### {Quality & Life of Your Product}

**IMPORTANT:** Routine maintenance will ensure quality and life of your long-term investment and protect animals from transmittable disease and infection

**IMPORTANT:** To safeguard your investment and rights to the Shor-Line's Warranty, it is important that you know factors that degrade stainless steel. Corrosion, pitting, and discoloration can occur if:

- Bleach, deodorizing agents, disinfectants, and sanitizers (chlorides, bromides, iodides and thiocyanates) stand wet on your product for a prolonged period (over two minutes).
- Bleach, deodorizing agents, disinfectants, and sanitizers are used frequently without being neutralized with clean water after each application.
- Acidic urine is not removed rapidly and surfaces are not washed thoroughly.
- Animal hair, holding urine and fluids, is allowed to collect around legs, hinges, casters, and welded wire intersections.
- Solutions like liquid bleach and antibacterials are allowed to evaporate and dry on surfaces.
- Particles from steel wool, metal fiber brushes, or metal scrapers (which are never recommended for cleaning stainless) get embedded in your product's surface, rust, and give the false impression that the stainless steel is defective.
- Your water source (local or well) is heavy in iron content and water is allowed to dry on your product.
- High levels of pollution (automotive exhaust, aggressive gases, soot, iron oxides, chemicals, etc.) or chloride (from sea or de-icing salts) exist in the environment and frequent, thorough cleaning (manual or rain-washing) does not occur.

### {Care & Cleaning Best Practices}

**IMPORTANT:** You play an essential role in safeguarding your investment and rights to Shor-Line's Warranty. We strongly recommend:

- Daily or weekly cleaning of stainless steel to prolong its longevity and value and to help you maintain its finish.
- Using the mildest cleaning procedure that will do the job effectively.
  - Ordinary waste deposits and fluids can usually be removed with soap and water. Rinse thoroughly with clear water and dry completely with a soft cloth to discourage water spotting.
  - Minor scale build up and some hard water spotting can be removed by washing with vinegar followed by a clearwater rinse and thorough drying.
  - If rubbing, brushing, or scraping is required, use only stainless steel wool, brushes with plastic, nylon or stainless steel fibers, and scrubbers made of plastic or nylon. Ordinary steel wool or steel brushes should never be used.
  - Stubborn deposits or tightly adhering debris may require a commercial cleaning product. Scouring powders will scratch stainless and should not be used. Always review label statements on cleaning products regarding use on metals.
  - Cleaning is not completed until fresh water has been used to thoroughly rinse all residue of salt, bleach, deodorizing agents, disinfectants, and sanitizers.
- Rubbing, drying, and polishing should always be in the direction of the product's grain so scratches will blend into the surface. Efforts opposite the grain may produce a scratched surface appearance.
- Rubbing a soft cloth moistened with olive oil, undiluted white or cider vinegar, or club soda may brighten stainless steel and help remove surface streaks and heat stains.
- Never use paint, lacquer, or varnish as a substitute or shortcut for properly caring and cleaning stainless steel.
- Never intentionally stamp or etch stainless surfaces.



## {Protecting Against Disease & Infection}

IMPORTANT: Shor-Line shares your commitment to health and safety. This information is intended help protect your animals, as well as, safeguard your investment.

- Bleaches can be harmful to animals. Bleaches can react with acids and release chlorine gas that is highly toxic.
- Because of its strong corrosive and toxic properties, bleach is diluted with water. The solution's potency (shelf-life) quickly degrades as the active chlorine dissipates with water. Animal care professionals, therefore, cannot know with certainty how effective the bleach solution is over time without implementing a formal monitoring system.
- If chemical deodorizers, disinfectants, or sanitizers are used, protect your animals from respiratory problems and infections by using clean water to wash residue from all surfaces and dry thoroughly.
- Clorox® Company warns: "We cannot recommend Clorox® to clean metal surfaces, including stainless steel. Prolonged or repeated exposure of metal to strong solutions of our product eventually can cause discoloration or corrosion. To reduce the possibility of metal discoloration, we suggest that exposure time to Clorox® be less than five minutes and that surfaces be rinsed well after application of the solution."

## Section 6: Services, Terms & Conditions, & Warranty

---

<https://www.midmark.com/service-support/terms-conditions>



# SHOR-LINE®

<i>toll free</i>	<i>local</i>	<i>fax</i>
800	913	913
444.1579	281.1500	281.5339

Shor-Line  
511 Osage Avenue  
Kansas City, KS 66105 U.S.A.  
[www.shor-line.com](http://www.shor-line.com)