

SHOR-LINE

now a part of midmark

Grooming Table PRODUCT GUIDE

ELITE GROOMING TABLE
SHOWN, MODELS VARY



LINKS TO VIDEO
INSTRUCTION
INCLUDED



Complies with: UL 499
CSA C22.2 No. 64-10

Save This Guide For Future Reference
PN: 006.3220.00

JUNE 2023

INTRODUCTION

Introduction

Thank you for purchasing Shor-Line products.

This Guide provides information regarding the installation, use, and care of your Shor-Line product. Keep this Guide in a safe and convenient place for reference.

For further questions, to purchase additional products, or to replace a lost or damaged Guide, please feel free to contact Shor-Line:

SHOR-LINE

511 Osage Ave.

Kansas City, Kansas 66105, USA

PHONE: 800.444.1579

LOCAL: 913.281.1500

FAX: 913.281.5339

EMAIL: guides@shor-line.com

WEB ADDRESS: shor-line.com

SHOR-LINE LIMITED

Vale Business Park Llandow,

Vale of Glamorgan CF71 7PF

United Kingdom

PHONE: +44 1446 77 20 41

FAX: +44 1446 77 36 68

EMAIL: quality@shor-line.co.uk

WEB ADDRESS: shor-line.co.uk

Shor-Line may provide instructions that supplement or supersede this Guide at any time. Contact Shor-Line to ensure the Guide is the latest version.

During installation, if a contradiction between this Guide, existing conditions, or local regulations arise, contact a Shor-Line representative before proceeding with installation. Discontinue use of the product if unsafe conditions become apparent.

The Elite, Big Top, Classic, and Shor-Pet Grooming Tables are also referred to as "Table" in this document. Components and options vary per model.

© Copyright 2023 Midmark Corporation, Miamisburg, Ohio USA. All rights reserved.



READ THIS GUIDE COMPLETELY BEFORE INSTALLATION AND USE AND THOROUGHLY UNDERSTAND AND FOLLOW ALL SAFETY INSTRUCTIONS.



WEAR PERSONAL PROTECTIVE EQUIPMENT, such as, but not limited to, eye protection, back support brace, and gloves during installation. Failure to do so could result in SERIOUS INJURY.



This product is intended to be used for animals only. Do not use for anything other than the intended purpose.

INTRODUCTION

Introduction	ii
--------------------	----

GENERAL INFORMATION

General information.....	1
Safety Alert Symbol	1
Personal Protective Equipment (PPE).....	1
Safety Warnings Included In This Guide	2
Safety Decal Locations.....	3

SECTION ONE, PRE-ASSEMBLY

Shipment Inventory And Inspection.....	4
Shipment Inventory.....	4
Shipment Inspection	4
Damage Reporting	4
Grooming Table Specifications	4
Component Parts List.....	5

SECTION TWO, ASSEMBLY PROCEDURES

Installation	6
Grooming Table Set-up.....	6
Adjustable Leveling Feet	6
Grooming Arm.....	7
Arm Installation	9

SECTION THREE, USE AND CARE

Use And Care.....	10
Safe Use Practices	10
Aftermarket Accessories	10
Lift Function	11
General Maintenance & Care	12
Maintenance Recommendations	12
Care Recommendations	12
Maintenance (Continued)	13
Foot Switch Sensitivity	13
Supplemental Instruction - Video Resources	14

SECTION FOUR, TERMS & CONDITIONS

Terms And Conditions	15
Contact Information.....	17

Refer to the Guide, images and content to assist with the installation of the product. Throughout the Guide, safety notices provide help for a successful installation.

SAFETY FIRST!

Shor-Line uses the following symbols and signal words to identify potential hazards or unsafe practices:

Safety Alert Symbol



Indicates a potential personal injury hazard exists. It is important to heed any safety warning information associated with this alert symbol.

Signal Words for Hazard Alerting Safety Messages



Indicates a hazardous situation which, if not avoided, WILL result in serious injury or death.



Indicates a hazardous situation which, if not avoided, COULD result in serious injury or death.



Indicates a hazardous situation which, if not avoided, COULD result in minor or moderate injury.

Important Information Symbol

NOTICE

Indicates information considered important but not directly hazard related.



Video Icon - Indicates additional information is available in a video format. Digital users can click the icon to view the video.

Always refer to the product guide for procedures and safety warnings.

Personal Protective Equipment (PPE)

PPE refers to protective clothing or other equipment designed to protect against injury. It is the responsibility of the client/installer to ensure all local and federal codes are adhered to during the installation and assembly of this product. Included is a list of PPE items suggested, but not limited to, protective equipment to help complete the installation safely.

- Back Support Brace
- Gloves
- Metatarsal, foot protection
- Eye Protection

Shor-Line makes no guarantee, implied or otherwise, that the information included in this Guide will be complete or failsafe, or that the information will prevent an injury from occurring. Standard measures described may not reflect the full extent of all steps that may need to be taken in any given emergency instance.



California Proposition 65 Warning

This product can expose you to chemicals including nickel (metallic), which are known to the state of California to cause cancer.

For more information go to: www.P65Warnings.ca.gov

Safety Warnings Included In This Guide

**DANGER**

Do NOT plug-in the power cord until the installation is complete and the equipment is ready for use. Ensure the power cord is free of kinks and not bound/pinched.

**WARNING**

*Do NOT leave patients on the table unattended. Failure to do so could result in **SERIOUS INJURY** or **DEATH**.*

***ALWAYS** maintain full control of patients when using the table or its operable functions. Failure to do so could result in **SERIOUS INJURY** or **DEATH**.*

*Do NOT use the table functions before checking to ensure the unit and surrounding area are clear of personnel, animals and obstructions. Failure to do so could result in **SERIOUS INJURY** or **DEATH**.*

*Keep personnel and animals clear of the foot operated lift control switch, unintentional table functions (movement) could occur. Failure to do so could result in **SERIOUS INJURY** or **DEATH**.*

**CAUTION**

This product is intended to be used for animals only. Do not use for anything other than the intended purpose.

*Keep hands clear of possible pinch areas during installation, which if not avoided, could result in **MINOR** to **MODERATE INJURY**.*

Do NOT exceed the weight limitations (250 lbs) of the table. Exceeding the weight limit may cause equipment failure.

*Stainless steel panels have sharp edges, stay clear of and **DO NOT TOUCH** exposed edges which, if not avoided, could result in **MINOR** to **MODERATE INJURY**.*

NOTICE

When not in use, store the table in its closed position and unplug the unit.

Safety Decal Locations



Grooming Table Safety Decals Location (Quantity And Label Types Vary Per Model)

Safety Decal List			
Item	Part #	Description	Qty
01	059.3220.01	SAFETY DECAL, Caution - Keep Hands Clear	2
02	059.3220.02	SAFETY DECAL, Caution - Electrical Hazard	1
03		SAFETY DECAL, Warning - Proposition 65	1

Read all warning, danger and caution decals before equipment is operated. Never use equipment if decals are missing, hidden, improperly placed, damaged or altered. Keep safety decals clean and legible.

Contact Shor-Line (800.444.1579) for replacement safety decals.

Shipment Inventory And Inspection

Shipment Inventory

Upon arrival, unpack and inspect the shipment to ensure it is complete and free of any damages that may have occurred during shipping. Compare the packing list with the shipment to ensure all parts/components have been received.

NOTICE

This Guide can be used to identify parts and their associated part numbers listed on the packing slip.

Shipment Inspection

While verifying the shipment contents, take a moment to inspect each component for damage. This should be done before the shipment is signed received and accepted.

If damaged components are apparent, follow the instructions within Shor-Line's Damage and Freight Procedures.

Damage Reporting

Follow the instructions within Shor-Line's Damage and Freight Procedures.

Contact Shor-Line (800.444.1579) immediately to expedite replacements or repairs.

Grooming Table Specifications

Part #	Description
903.3220.42*	Elite Grooming Table, 120vac Lift Actuator, Shor-Line Blue**
903.3220.44	Elite Grooming Table, Hydraulic Lift Actuator, Shor-Line Blue**
903.3220.63*	Big Top Grooming Table, 120vac Lift Actuator, Shor-Line Blue**
903.3220.70	Big Top Grooming Table, Hydraulic Lift Actuator, Shor-Line Blue**
903.3220.27*	Classic Grooming Table, 120vac Lift Actuator, Shor-Line Blue**
* 230vac Electric Lift Actuator Available ** Silver Vein, Plum Perfect, Screamin' Green, Burnt Orange and Electric Pink powder-coated frames are available. Contact Shor-Line for table model and finish options at (800.444.1579)	

Grooming Table Electric Actuator	
24v DC Motor	Max Height (Floor to table top surface): 41 inches (1041mm)
2.7amp Max at 250lbs (1112 Newton) load	Min Height (Floor to table top surface)? 20 inches (508)
Max Shaft Travel: 6 inches (152mm)	Elapsed time of travel min to max height (no load): 14 sec
120v AC power cord (120v model)	Elapsed time of travel max to min height (no load): 13 sec
230v AC power cord (230v model)	Min Dimensions: 42.25"H (1073mm), 46.5"L (1184mm), 27.43"W (697mm)
DC control module mounted to actuator	Max Dimensions: 80"H (2032mm), 54.5"L (1384mm), 27.43"W (697mm)

Grooming Table Hydraulic Actuator	
Manual Hydraulic Pump, Pedal Actuated	Max Height (Floor to table top surface): 41 inches (1041mm)
Max at 250lbs (1112 Newton) Load	Min Height (Floor to table top surface)? 20 inches (508)
Max Shaft Travel: 6 inches (152mm)	Elapsed time of travel min to max height (no load): 14 sec
	Elapsed time of travel max to min height (no load): 13 sec

SECTION ONE

Component Parts List

NOTE: Big Top table shown.

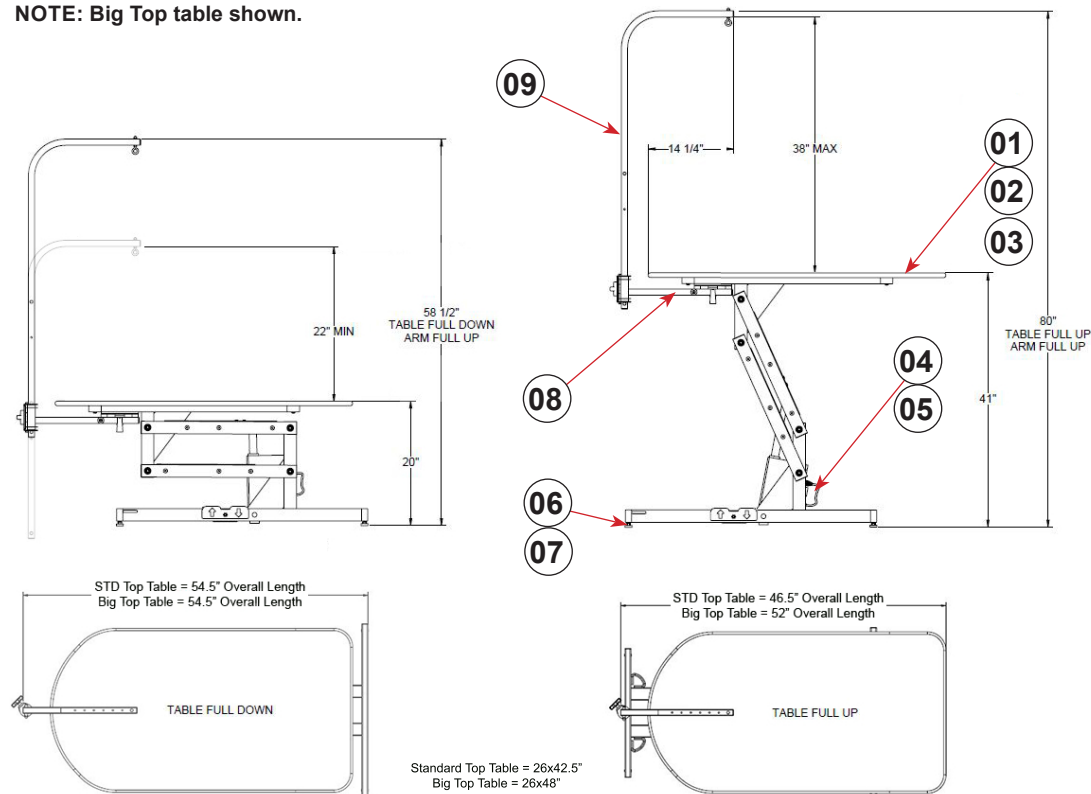


FIGURE 1.1 - Grooming Table Dimensions and Parts List (Big Top Table Shown)

Item	Part #	Description
01		Standard Grooming Table, 26x42.5" Top
02		Big Top Grooming Table, 26x48" Top
03		Shor-Pet Grooming Table, 26x42.5" Top
04	089.0013.03	Electric Lift Actuator (120vac)* (Not optional for the Shor-Pet model)
05	181.0068.00	Hydraulic Lift Actuator
06	062.3715.20	Leveling Foot Screw
07	051.0000.00	Leveling Foot Scuff Pad
08		Grooming Arm Swivel Bracket (Not optional for the Shor-Pet model)
	• 058.3008.00	• Tapered Handle, Swivel Bracket
	• 085.1018.00	• Handle Spring, Swivel Bracket
	• 310.1066.00	• Handle Plunger, Swivel Bracket
09	803.3221.04	Grooming Arm Assembly (Not optional for the Shor-Pet model)
	• 181.3220.00	• Grooming Arm Scuff Ball
	• 300.2451.00	• Tube Insert Nut, Scuff Ball
	• 062.2520.03	• 1/4x3/4" Long Shoulder Bolt, Scuff Ball
	140.0001.32	Grooming Loop (Not Shown)

* 230vac Electric Lift Actuator available.

Contact Shor-Line for table model and finish options at (800.444.1579)

Installation

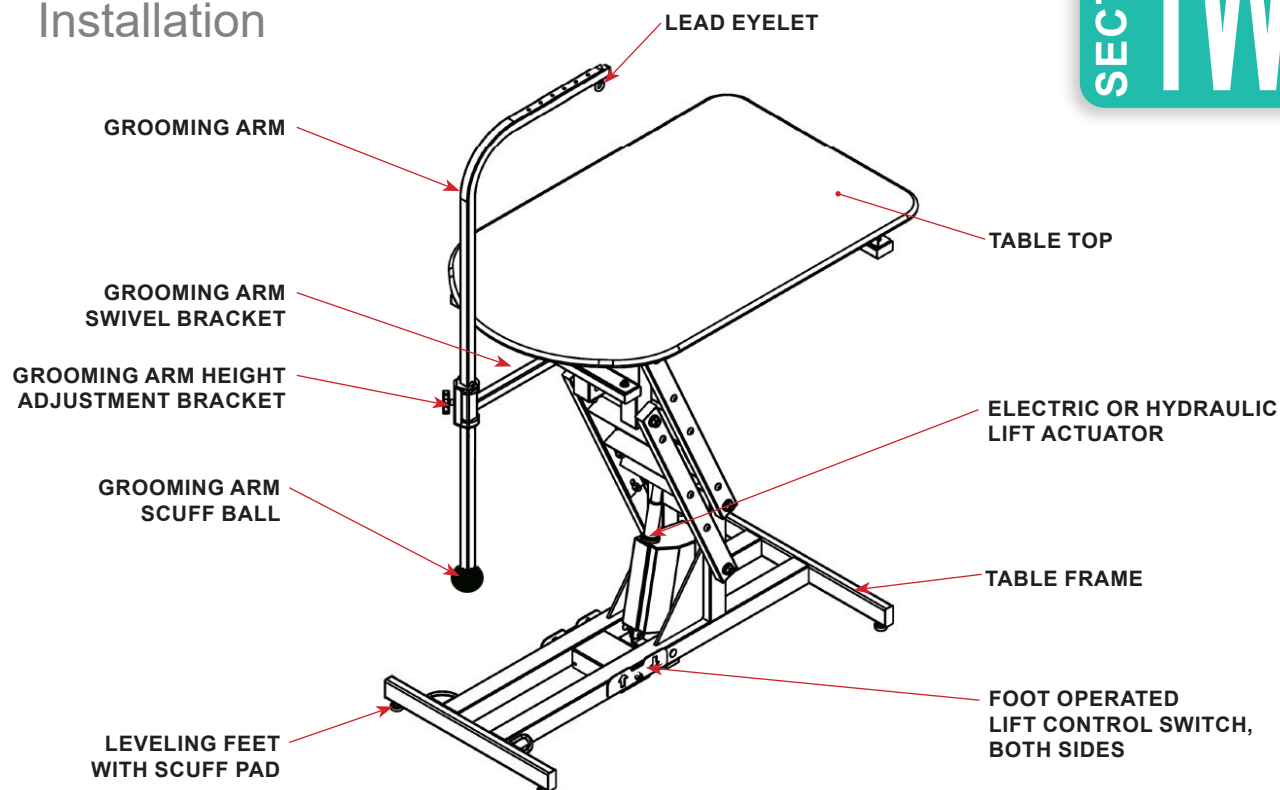


FIGURE 2.1 - Grooming Table (Elite Table w/Electric Actuator Shown, Components/Models Vary)

Grooming Table Set-up

Adjustable Leveling Feet

Grooming tables equipped with an electric powered lift actuator must be located with an electrical outlet near the unit, dedicated. Do NOT use extension cords to provide power for the table.



WARNING

Enlist help and USE SAFE LIFTING PRACTICES during assembly, installation and moving equipment.



CAUTION

Keep hands clear of possible pinch areas during installation, which if not avoided, could result in MINOR to MODERATE INJURY.



NOTICE

Do NOT lift or move the table by the platform. Always move the table by the base frame using enough personnel and safe lifting procedures.

STEP 1: Lay the table on its side to install the adjustable leveling feet and scuff pads.

STEP 2: Thread the leveling screws (feet) into the bottom of the frame. Leave 1/2" of screw threading exposed. See FIGURE 2.2

STEP 3: Slip a scuff pad over each leveling foot.

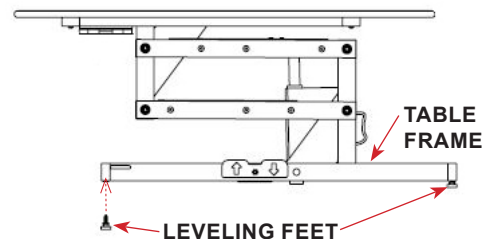


FIGURE 2.2 - Adjustable Leveling Feet

SECTION TWO

Grooming Arm



Click the Video Icon to view an instructional video on the [Arm Installation](#).

Always refer to the product guide for procedures and safety warnings.

The Elite Grooming Table is equipped with a shepherd's hook shaped grooming arm that can be used to secure a lead.

NOTICE

The grooming arm must be installed before installing the lower slide stop and scuff ball.

The grooming arm can be adjusted vertically by loosening the height adjustment bracket. There are slide stop pins located at the bottom and halfway up the grooming arm to prevent vertical arm movement beyond these limits.

Several holes along the top/end of the arm are used for the leash eyelet. Remove the nut and reposition the eyelet to the desired location as needed.

The swivel bracket controls lateral movement. Disengage the lock handle by pulling downward to rotate the arm around the end of the table, there are five locked positions. Ensure the swivel bracket is locked in position before securing animals to the grooming arm. See FIGURE 2.3

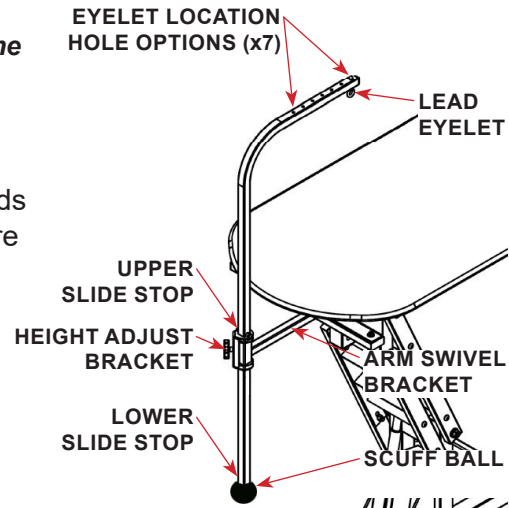


FIGURE 2.3 - Adjustable Grooming Arm



Adjustable Grooming Arm

Click the Video Icon to view an instructional video on [Adjusting The Arm](#).

Always refer to the product guide for procedures and safety warnings.

The grooming arm can be installed with the table on its side. The following instructions provide direction for installing the arm with the table upright with the top fully raised to its uppermost position.

To install the grooming arm into the height adjustment bracket the lower slide stop must be removed from the grooming arm. Remove the acorn nut, spacers and bolt. If the arm is equipped with post end plug, remove before installing the grooming arm.



WARNING Enlist help and **USE SAFE LIFTING PRACTICES** during assembly, installation and moving equipment.



CAUTION Keep hands clear of possible pinch areas during installation, which if not avoided, could result in **MINOR to MODERATE INJURY**.

STEP 1: Slide the height bracket over the end of the arm swivel bracket if it is not already installed. Loosen the height adjustment bracket knob. See FIGURE 2.4

STEP 2: Slide the straight long end of the grooming arm down into and through the top of the height adjust bracket. Ensure the top of the shepherd's hook is facing towards the center of the table. Allow the arm to rest on the height control bracket, tighten the knob to help aid in the scuff ball installation.

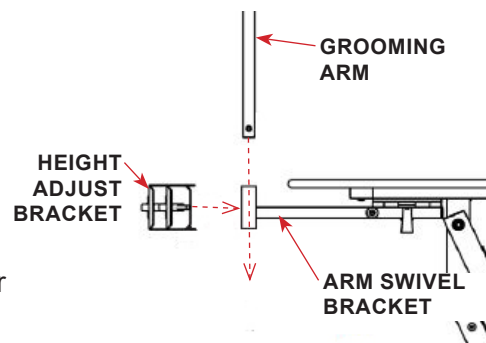


FIGURE 2.4 - Height Adjust Bracket

Scuff Ball

STEP 3: Thread the shoulder bolt through the bottom hole of the scuff ball and into the insert nut (teeth towards bolt) until the insert nut is just below the top of the scuff ball. See FIGURE 2.6

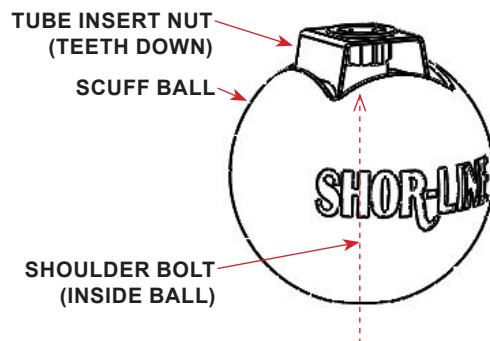


FIGURE 2.6 - Tube Insert Nut

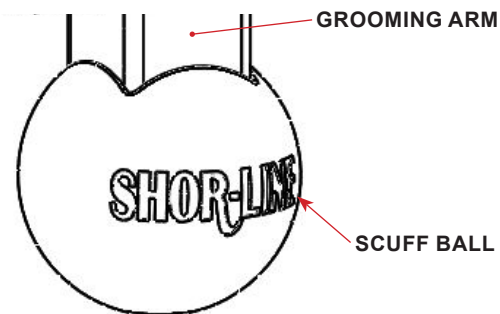


FIGURE 2.7 - Fixed Grooming Arm

STEP 4: Carefully align and insert the scuff ball assembly onto the end of the grooming arm. Apply pressure and work (wiggle) until the ball is seated to the bottom of the arm. See FIGURE 2.7

STEP 5: Tighten (tool provided) the shoulder bolt to secure the scuff ball in place. Do NOT over-tighten.

NOTICE

The shoulder bolt head should be recessed into the scuff ball. If the bolt head is exposed it can cause damage to the floor surface.

Lower Slide Stop

The lower slide stop pin is NOT used on some models of the grooming arm. The stop pin holes may be blocked by the scuff ball. Omit the stop pin if there is interference with the scuff ball assembly.

STEP 6: Slide one of the spacers onto the lower slide bolt and insert through the lower hole on the arm. See FIGURE 2.8

STEP 7: Insert the second spacer onto the screw and secure by tightening the acorn nut onto the bolt. Do NOT over-tighten.

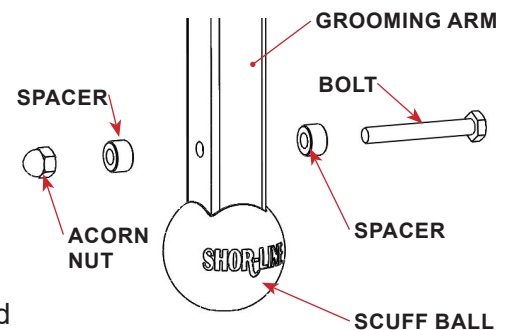


FIGURE 2.8 - Lower Slide Stop Pin

SECTION TWO

Fixed Grooming Arm (Optional Accessory)

Shor-Line offers two grooming table top sizes; Standard model measuring 26x42.5" long and Large measuring 26x48" long grooming tables that are equipped with the framing and mounting holes for the Fixed Grooming Arm. Ensure the bracket is the correct size for the table. The standard arm bracket will NOT fit on a large table.

NOTICE

The Fixed Grooming Arm is not compatible with the Shor-Pet grooming table.

The fixed grooming arm is mounted on the opposite end of the table. The arm bracket is bolted to the underside of the table top frame. The height can be adjusted using the same procedures as the adjustable grooming arm.

To install the fixed grooming arm into the height adjustment bracket the lower slide stop must be removed from the grooming arm. Remove the acorn nut, spacers and bolt. If the arm is equipped with post end plug, remove before installing the fixed grooming arm. See FIGURE 2.9

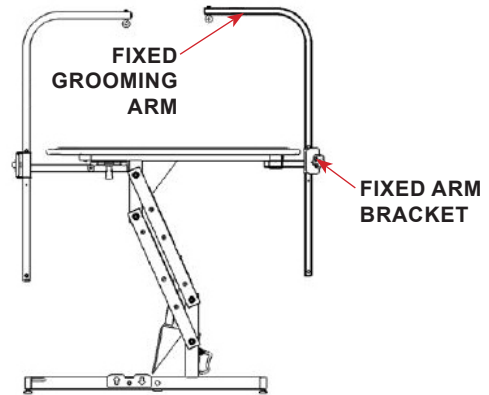


FIGURE 2.9 - Fixed Grooming Arm

WARNING

Enlist help and USE SAFE LIFTING PRACTICES during assembly, installation and moving equipment.

CAUTION

Keep hands clear of possible pinch areas during installation, which if not avoided, could result in MINOR to MODERATE INJURY.

STEP 1: Secure the fixed arm bracket to the underside of the table top frame using the hardware provided. See FIGURE 2.10

STEP 2: Slide the height bracket over the end of the arm swivel bracket if it is not already installed. Loosen the height adjustment bracket knob.

STEP 3: Slide the straight long end of the grooming arm down into and through the top of the height adjustment bracket. Ensure the top of the shepherds hook is facing towards the center of the table. Allow the arm to rest on the height control bracket, tighten the knob to help aid in the scuff ball installation.

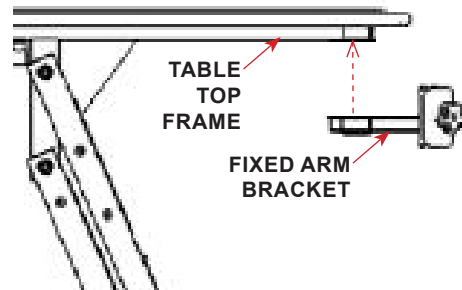


FIGURE 2.10 - Fixed Arm Bracket

STEP 4: Follow the adjustable arm procedures to complete the installation of the fixed grooming arm, lower slide stop pin and the scuff ball.

Arm Installation

The Elite Grooming Table is equipped with a shepherds hook shaped grooming arm that can be used to secure a lead.

Click the Video Icon to view an instructional video on the [Arm Installation](#).

Always refer to the product guide for procedures and safety warnings.



Use And Care

Safe Use Practices

Always use safe practices during loading and unloading patients from the grooming table. All procedures should be created by the facility owner and followed. Common sense and being aware of safety concerns should always drive procedures. Below is a list of recommendations and situational conditions that require attention, heed all warnings.

- Unplug the unit before conducting any maintenance or cleaning procedures. Do NOT allow any personnel under or around the unit when plugged into a power outlet.
- Do NOT operate the grooming table before checking to ensure the unit and surrounding area is clear of personnel, animals and obstructions.
- Do NOT operate the grooming table on uneven flooring. Adjust leveling feet to ensure the unit is stable and the patient is secure during operation.
- Do NOT sit, stand or lean on the table. Animal use ONLY.



DANGER Do NOT plug-in the power cord until the installation is complete and the equipment is ready for use. Ensure the power cord is free of kinks and not bound/pinched.



WARNING Do NOT leave patients on the table unattended. Failure to do so could result in **SERIOUS INJURY or DEATH.**

ALWAYS maintain full control of patients when using the table or its operable functions. Failure to do so could result in **SERIOUS INJURY or DEATH.**

Do NOT use the table functions before checking to ensure the unit and surrounding area are clear of personnel, animals and obstructions. Failure to do so could result in **SERIOUS INJURY or DEATH.**

Keep personnel and animals clear of the foot operated lift control switch, unintentional table functions (movement) could occur. Failure to do so could result in **SERIOUS INJURY or DEATH.**

NOTICE

When not in use, store the table in its closed position and unplug the unit.

Adjustable Grooming Arm, Shepherds Hook



The adjustable grooming arm and shepherds hook can be adjusted to rotate around the end of the table and the shepherds hook can be raised or lowered as required for patient height.

Click the Video Icon to view an instructional video on the [Adjusting The Arm](#).

Always refer to the product guide for procedures and safety warnings.

Aftermarket Accessories

Use of aftermarket accessories (not Shor-Line) such as clamp-on grooming arms and other restraint systems is NOT recommended. Some aftermarket accessories may put an unintentional load on the table that exceeds its design specifications. Damage caused by aftermarket accessories attached to Shor-Line tables may void product warranty.

NOTICE

Do NOT use aftermarket accessories on Shor-Line products.

SECTION THREE

Lift Function

The grooming table is equipped with either an electric or hydraulic lift feature to raise and lower the table platform approximately 21 inches.



Do NOT use the table functions before checking to ensure the unit and surrounding area are clear of personnel, animals and obstructions. Failure to do so could result in SERIOUS INJURY or DEATH.

Keep personnel and animals clear of the foot operated lift control switch, unintentional table functions (movement) could occur. Failure to do so could result in SERIOUS INJURY or DEATH.

Electric Lift

The electric lift is operated by a foot activated pedal. Depress the pedal to raise and lower the platform. Release the pedal to stop movement. See FIGURE 3.1, 3.2 & 3.3

The electric lift is capable of lifting up to 250 lbs of weight on the table top surface.



FIGURE 3.1 - Electric Lift

NOTICE

Do NOT exceed the weight limitations (250 lbs) of the table. Exceeding the weight limit may cause equipment failure.

When not in use, store the grooming table in the down position and unplug the unit.

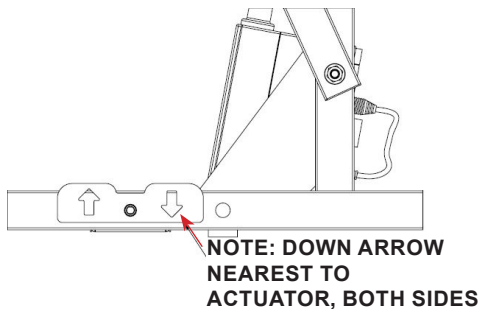


FIGURE 3.2 - Lift Pedal

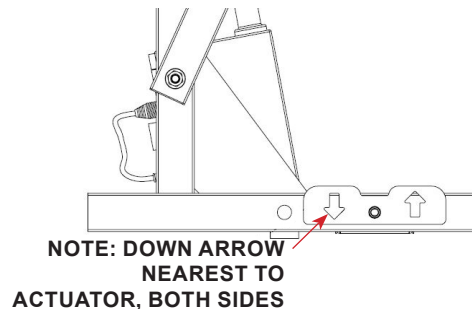


FIGURE 3.3 - Lift Pedal (Opposite Side)

Hydraulic Lift

The Hydraulic lift is operated by a foot activated pedal. Depressing the pedal lifts the table surface inches per pump. To lower the table, fully depress and hold the pedal until the top reaches the desired height. Release the pedal to stop movement. See FIGURE 3.4

NOTICE

Do NOT exceed the weight limitations (250 lbs) of the table. Exceeding the weight limit may cause equipment failure.

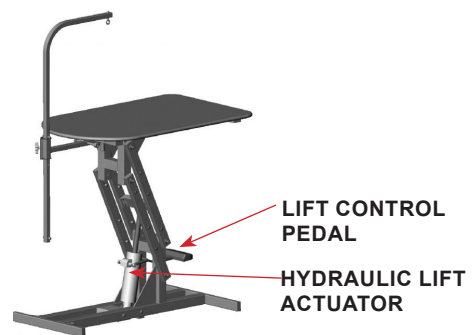


FIGURE 3.4 - Hydraulic Lift

General Maintenance & Care

Maintenance Recommendations

NOTICE *Routine maintenance will extend the quality and life of Shor-Line products.*

It is the owners responsibility to set-up scheduled maintenance programs, depending on use of the equipment. Scheduled preventive maintenance should include, but not limited to daily or weekly inspections and maintenance of products to prolong its longevity and to help maintain proper product functions.

- Follow manufacturers cleaning and maintenance recommendations outlined in product literature provided for components not included in these recommendations. Configurations/component parts may vary.
- Check alignment of operable panels, doors and components to ensure operation is free of obstructions. Adjust as required.
- Check any battery compartments to ensure the batteries are not failing, corroding the contacts or damaging the equipment. A pencil eraser can be used to clean electrical contacts in battery compartments.
- Check any wiring for kinks or exposed inner wiring. If damage exist, stop equipment use and contact Shor-Line Technical Services for direction.
- Check any hydraulic lift components for leaking fluid. If fluid is present, stop equipment use and contact Shor-Line Technical Services for direction. Do NOT attempt to repair hydraulic leaks.

Care Recommendations

NOTICE *Routine product care will extend the quality and life of Shor-Line products and aids in protecting animals from transmittable diseases and infections.*

It is the owners responsibility to set-up scheduled cleaning programs, depending on use of the equipment. Scheduled preventive cleaning should include, but not limited to daily or weekly inspections and cleaning of products to prolong its longevity and to help maintain finishes.

Safe Cleaning Practices

- Use the mildest cleaning procedure that will complete the job effectively. Ordinary waste deposits and fluids can usually be removed with soap and water, using a soft cloth or sponge. Rinse thoroughly with clear water and dry completely with a soft cloth to discourage hard-water spotting.
- Minor scale build up and some hard water spotting can be removed by washing with a vinegar diluted mixture followed by a clear water rinse and thorough drying.
- Bleach, deodorizing agents, disinfectants, and sanitizers can corrode stainless steel, thoroughly rinse all surfaces treated with these chemicals with a clear water rinse and dry with a soft cloth.
- If scrubbing is required, use only polymer or nylon fiber products made for use with polymer or nylon materials.
- Always rinse with clear water and dry of all surfaces treated with cleaning, sterilization solutions.

Unsafe Cleaning Practices (NOT Recommended)

- Do NOT use a dry cloth or wipe clear polymer surfaces which can scratch if dust/dirt is wiped with the hand or dry cloth.
- Polymer materials can discolor if exposed to sunlight, ultraviolet rays. Avoid direct sun exposure.
- Do NOT use ammonia or bleaches to clean polymer surfaces.
- Do NOT allow fluids to accumulate, puddle without removing and completely drying the surfaces and components. Standing water/fluids are a hazard and can cause damage to component materials.
- Do NOT use scouring powders that will scratch polymer finishes.

NOTICE *Steel wool or steel brushes should never be used to clean stainless steel or polymer surfaces, avoid abrasive cleaning techniques/supplies.*

SECTION THREE

Maintenance (Continued)

Lubricating Joints

The lift frame connection joints may begin to squeak over time. To properly lubricate the frame joints, loosen the connection bolts by 1/4 turn. Spray lubricate between the frame members. Do NOT re-tighten the connection bolts.

Click the Video Icon to view an instructional video for [Maintenance-Lubricating Joints](#).

Always refer to the product guide for procedures and safety warnings.

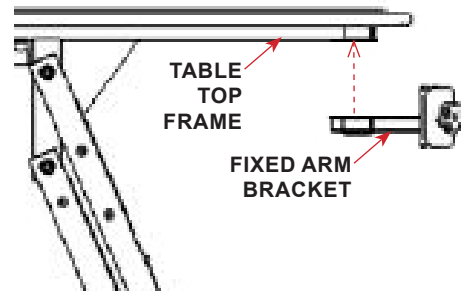


FIGURE 2.10 - Fixed Arm Bracket

Foot Switch Sensitivity

Spring Plate Adjustment

NOTICE

Routine product care will extend the quality and life of Shor-Line products.

Grooming tables equipped with electric lift actuators are operated by a foot operated pedal. Inside the pedal control box there is a spring plate that may loosen due to frequent use of operation for extended periods of time.

If the foot operated pedal appears to be less responsive, the spring plate screws may be tightened to elevate pedal sensitivity issues. See FIGURE 3.5

WARNING

Enlist help and USE SAFE LIFTING PRACTICES during assembly, installation and moving equipment.

CAUTION

Keep hands clear of possible pinch areas during installation, which if not avoided, could result in MINOR to MODERATE INJURY.

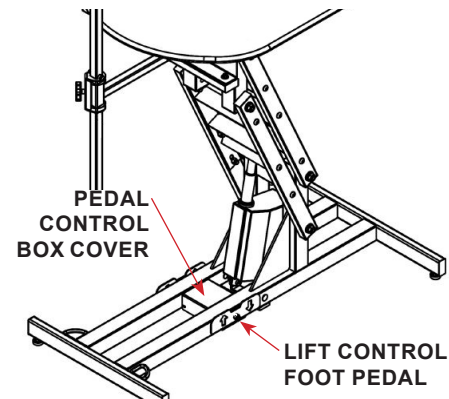


FIGURE 3.5 - Lift Switch Control Box

Click the Video Icon to view an instructional video for [Adjusting The Footswitch](#).

Always refer to the product guide for procedures and safety warnings.

Fully extend (raise) the table top and unplug the unit before continuing with the procedures. The pedal control box cover and the spring plate adjustment screws are accessible from the topline of the unit. The unit can be set on its side for the procedures if preferred.

STEP 1: Secure the fixed arm bracket to the underside of the table top frame using the hardware provided. See FIGURE 2.10

STEP 2: Tighten the two center screws with a 4mm Allen wrench.

DANGER

Do NOT plug-in the power cord until the installation is complete and the equipment is ready for use. Ensure the power cord is free of kinks and not bound/pinched.

WARNING

Do NOT use the table functions before checking to ensure the unit and surrounding area are clear of personnel, animals and obstructions. Failure to do so could result in SERIOUS INJURY or DEATH.

Keep personnel and animals clear of the foot operated lift control switch, unintentional table functions (movement) could occur. Failure to do so could result in SERIOUS INJURY or DEATH.

NOTICE

When not in use, store the table in its closed position and unplug the unit.

Component Replacement


If hydraulic or electric lift actuator replacement is required: Installation and repair of table components must be performed by a qualified electrician/contractor. Failure to comply will result in a void of Shor-Line warranties.




Click the Video Icon to view an instructional video for [Replacing The Actuator](#).

Always refer to the product guide for procedures and safety warnings.


Supplemental Instruction - Video Resources

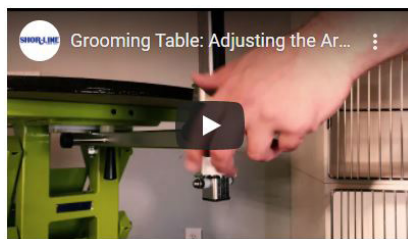


[ARM ADJUSTMENT](#)

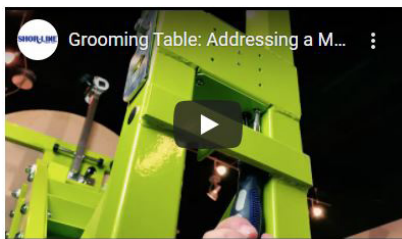

[FOOT SWITCH ADJUSTMENT](#)

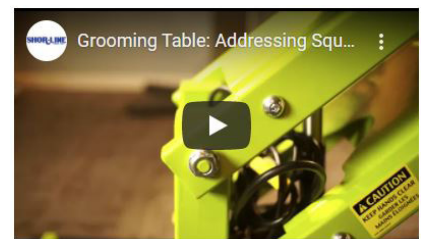


[MAINTENANCE - LUBRICATING JOINTS](#)

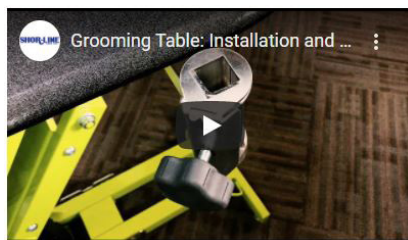


[ARM INSTALLATION](#)




[TABLE TOP RISER INSTALLATION](#)

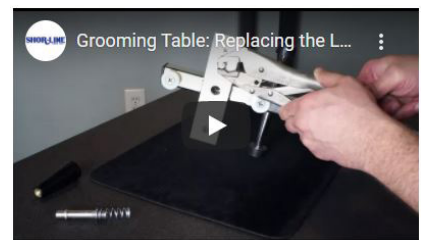


[LOCK KNOB REPLACEMENT](#)

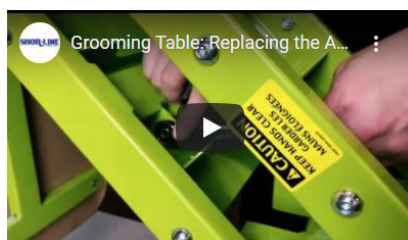

[ACTUATOR REPLACEMENT](#)

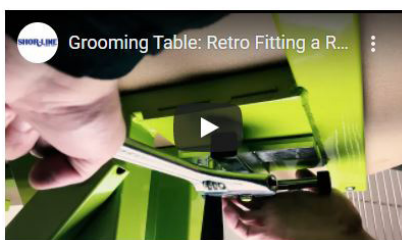

[RETROFIT ROTATING ARM](#)

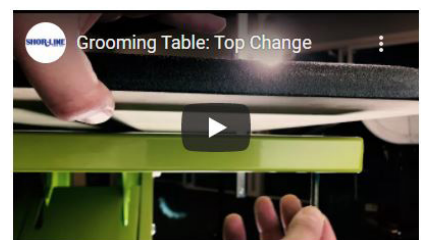

[TABLE TOP REPLACEMENT](#)

ALWAYS REFER TO THE PRODUCT GUIDE FOR PROCEDURES AND SAFETY WARNINGS

SECTION FOUR

Terms And Conditions

<https://www.midmark.com/service-support/terms-conditions>



SHOR-LINE

now a part of midmark

SHOR-LINE

511 Osage Ave.

Kansas City, Kansas 66105, USA

PHONE: 800.444.1579

LOCAL: 913.281.1500

FAX: 913.281.5339

EMAIL: guides@shor-line.com

WEB ADDRESS: SHOR-LINE.com

SHOR-LINE LIMITED

Vale Business Park Llandow,

Vale of Glamorgan CF71 7PF

United Kingdom

PHONE: +44 1446 77 20 41

FAX: +44 1446 77 36 68

EMAIL: quality@shor-line.co.uk

WEB ADDRESS: SHOR-LINE.co.uk