



A guide for:
Cage End Panels

SHOR-LINE

{Save This Guide For Future Reference}

PN: 006.9020.03
Updated: 06.03.14



Introduction

Thank you for purchasing Shor-Line products.

We hope this guide answers any questions you might have in regards stainless steel care and cleaning. If you can't find the answers to your questions, have further questions or would like to purchase additional products, please contact us at 800.444.1579 or 913.281.1500. If in Europe phone +44 1446 77 20 41.

Consider keeping this guide in a safe and convenient place for future assistance.



WARNING California Proposition 65 Warning

This product can expose you to chemicals including nickel (metallic), which are known to the state of California to cause cancer.

For more information go to: www.P65Warnings.ca.gov

{Shor-Line}

511 Osage Ave.
Kansas City, Kansas 66105, USA
phone 800.444.1579 fax 913.281.5339
email guides@shor-line.com
www.shor-line.com

{Shor-Line Limited}

Vale Business Park
Llandow,
Vale of Glamorgan CF71 7PF
United Kingdom
phone +44 1446 77 20 41 fax +44 1446 77 36 68
email quality@shor-line.co.uk
www.shor-line.co.uk

▲WARNING

Shor-Line products are designed for animal use only.




Installing Cage End Panels

{Before Installation}

- Remove grease, grit, and grime from the side of all End Cages (right) using the alcohol wipes we provided.

IMPORTANT: The cages on either end of the assembly must be thoroughly clean and dry for our two-way adhesive tape to hold. See Fig. 1.

Fig. 1

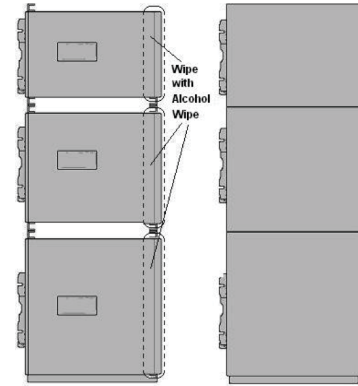


{Cage End Panel Installation}

Step 1

Loosen all External Screws connecting outside End Cage Hinges, Latches, and Trim Plates so End Panels can be inserted behind them. See Fig. 2.


Fig. 2

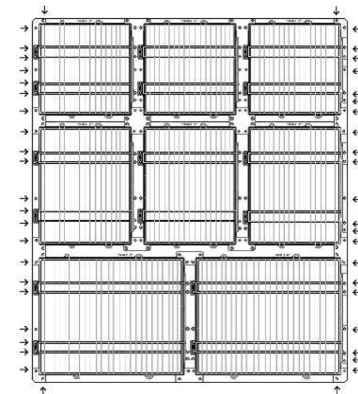

Step 2

Peel the white protective covering off of the Cage End Panels. Begin with the Cage on the end of the Bottom Row and locate the appropriate-sized End Panel. Panels vary in height but have slots cut into their front frames to align with Hinge and Latch Screws. See Fig. 3.

Fig. 3

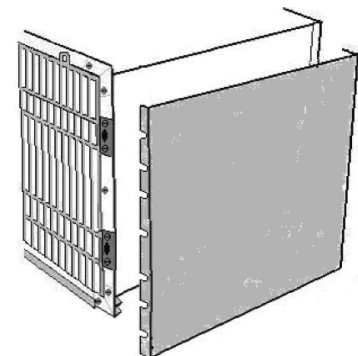




Step 3

Remove the adhesive tape backing from the End Panel's back lip. See Fig. 4.

Fig. 4

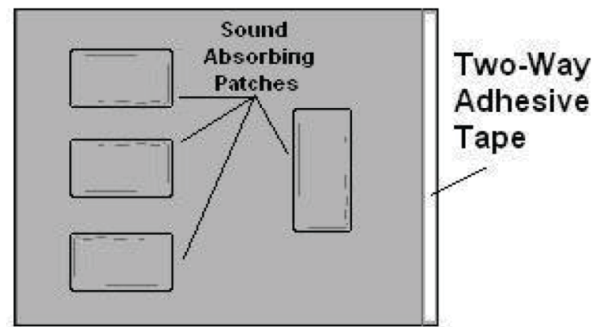

Step 4

Position End Panel onto Cage so Panel's slotted front frame fits beneath the Cage's Hinge and/or Latch Screws and the two corner Trim Plate Screws. Fit the End Panel directly over the outer-side of the Cage. See Fig. 5.


Step 5

Press all four sides of the panel firmly against the Cage side so the two become connected. Provide pressure for about 60 seconds. Repeat for all End Panels.

Step 6

When all are installed, tighten all screws (refer to Step 1) on your kennel system securely.

Fig. 5

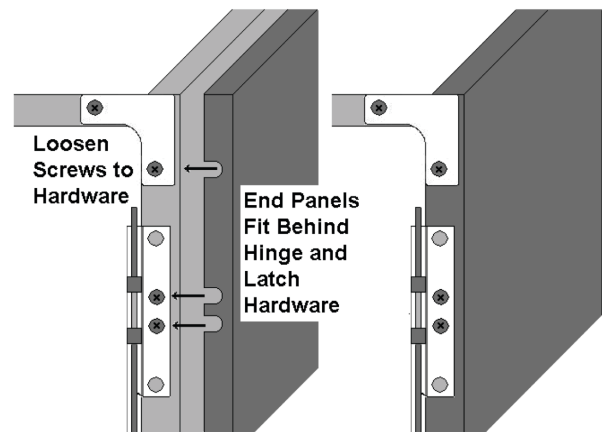

<https://www.midmark.com/service-support/terms-conditions>



SHOR-LINE®

<i>toll free</i>	<i>local</i>	<i>fax</i>
800	913	913
444.1579	281.1500	281.5339

Shor-Line
511 Osage Avenue
Kansas City, KS 66105 U.S.A.

www.shor-line.com