



A guide for:
Squeeze Restraint

SHOR-LINE®

⚠WARNING This product is for animal use only. We do not condone this product for other uses.

⚠WARNING To reduce the risk of injury, the user must read and understand this guide before installation of this product.

{Save This Guide For Future Reference}

PN: 006.9240.20
Updated: 10.08.13



Introduction

Thank you for purchasing Shor-Line products.

We hope this guide answers any questions you might have in regards to use, care or installation. If you can't find the answers to your questions, have further questions or would like to purchase additional products, please contact us at 800.444.1579 or 913.281.1500. If in Europe, phone +44 1446 77 20 41.

Consider keeping this guide in a safe and convenient place for future assistance.

For your safety, please follow the instructions on these labels:

▲CAUTION ▲WARNING

▲WARNING This product is for animal use only. We do not condone this product for other uses.

▲WARNING To reduce the risk of injury, the user must read and understand this guide before installation of this product.

{Shor-Line}
511 Osage Ave.
Kansas City, Kansas 66105, USA
phone 800.444.1579 fax 913.281.5339
email guides@shor-line.com
www.shor-line.com

{Shor-Line Limited}
Vale Business Park
Llandow,
Vale of Glamorgan CF71 7PF
United Kingdom
phone +44 1446 77 20 41 fax +44 1446 77 36 68
email quality@shor-line.co.uk
www.shor-line.co.uk



WARNING California Proposition 65 Warning

This product can expose you to chemicals including nickel (metallic), which are known to the state of California to cause cancer.

For more information go to: www.P65Warnings.ca.gov

▲WARNING

Shor-Line products are designed for animal use only.

{Contents}

Section 1: Unpacking & Inspecting Shipment	4
Unpacking & Inspecting Steps	4
Parts and Hardware for Assembly List	5
Section 2: Squeeze Restraint Assembly Instructions	6
Section 3: Services, Terms & Conditions, Warranty	
Terms & Conditions	8

- | | |
|---|-------------------|
| • Squeeze Restraint Cage Insert , 24" x 18" | Item #924.0200.00 |
| • Squeeze Restraint Cage Insert , 24" x 24" | Item #924.0200.01 |
| • Squeeze Restraint Cage Insert , 24" x 30" | Item #924.0200.02 |
| • Squeeze Restraint Cage Insert , 30" x 30" | Item #924.0200.03 |



Section 1: Unpacking & Inspecting Shipment

{Unpacking & Inspecting Steps}

Step 1

Locate the Packing Slip that describes the parts and quantities shipped. The parts for your Squeeze Restraint should be carefully examined upon arrival for possible damage in shipment. If any damage is found, notify Shor-Line immediately at 800-444-1579.

Step 2

Review Shor-Line's warranty information at: www.shor-line.com

Step 3





Remove and separate all contents from the shipping containers. Use the Packing Slip to ensure nothing has been lost.

Step 4

Check content for concealed damage.



{Squeeze Restraint Parts & Hardware for Assembly List}

QTY	PART	DESCRIPTION/PART NUMBER
2		724.0200.15 WELDMENT,SQUEEZE RESTRAINT 18.5 HANDLE 724.0200.16 WELDMENT,SQUEEZE RESTRAINT 15.5 HANDLE 724.0200.17 WELDMENT,SQUEEZE RESTRAINT 11.689 HANDLE
1		724.0200.11 WELDMENT, 24X18 SQUEEZE DOOR 724.0200.12 WELDMENT, 24X24 SQUEEZE DOOR 724.0200.13 WELDMENT, 24X30 SQUEEZE DOOR 724.0200.14 WELDMENT, 30X30 SQUEEZE DOOR
8		061.2505.10 .250-28 x .500 TRUSS HEAD SS PHILLIPS SCREW
8		063.2500.01 .250-28 HX NUT NYLOCK SS

Squeeze Restraints are available in four sizes depending upon the size of cage for the intended use. The parts list will vary based on size purchased. Some assembly is required but the steps will be the same for all sizes.



Section 2: Squeeze Restraint Assembly Instructions

Step 1

The squeeze door is supplied with the door attachments already welded in place. Using the .250-28 stainless steel phillips screws and .250-28 nylock nuts, attach the handle weldments to the squeeze door. The hardware should be placed in all four places allotted for each handle (two on top and two on bottom). See Fig. 1 and 2.

NOTE: Failure to install all hardware could result in a malfunction of the squeeze restraint.

Once the squeeze restraint has been assembled you will be able insert the unit into your cage.

Fig. 1



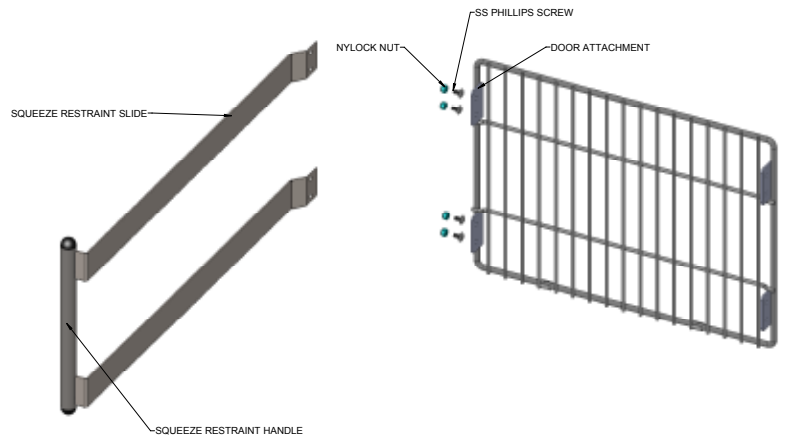


Fig. 2

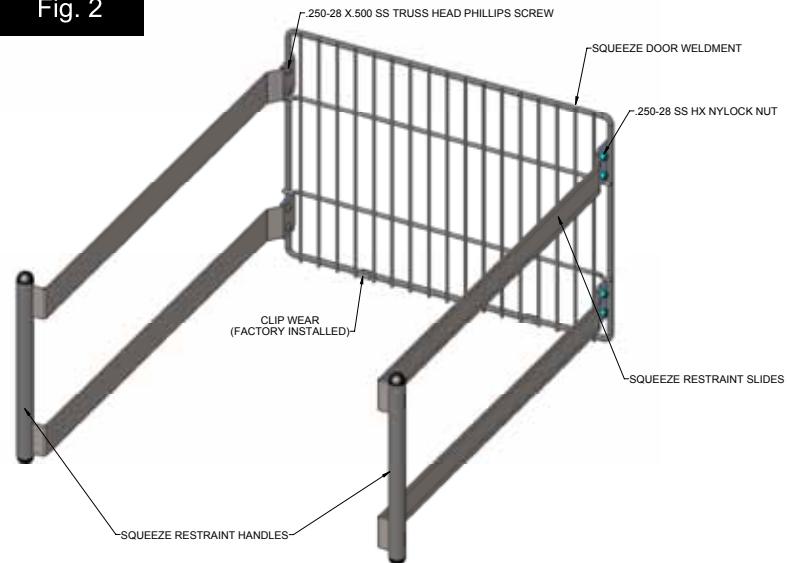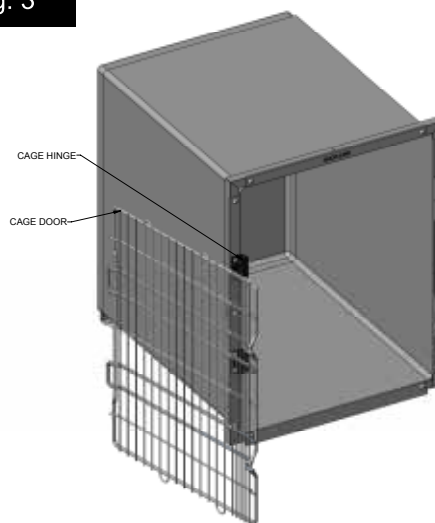



Step 2

Open the cage door 90 degrees, lift up and remove. See Fig. 3.

Fig. 3**Step 3**

Place Squeeze Restraint inside the cage. Resting the Squeeze Restraint slide on the top of the cage door hinge will allow you to replace the cage door. See Fig. 4 and 5.

Once you have replaced the cage door you will be able to move the squeeze restraint in and out as needed.

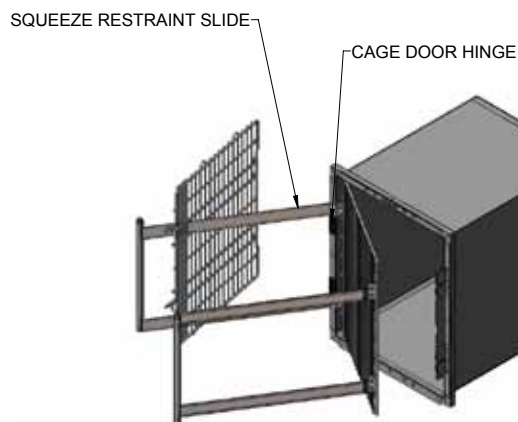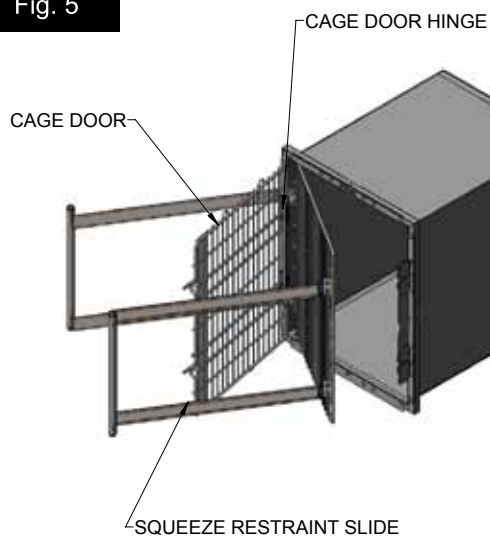Fig. 4**Fig. 5**



Section 3: Services, Terms & Conditions

<https://www.midmark.com/service-support/terms-conditions>



SHOR-LINE®

<i>toll free</i>	<i>local</i>	<i>fax</i>
800	913	913
444.1579	281.1500	281.5339

Shor-Line
511 Osage Avenue
Kansas City, KS 66105 U.S.A.

www.shor-line.com